F. BOSCH

MEDIZINTECHNIK
GERMANY

Instructions For Use

Product: Ultrasound probe cover with lubricant, sterile

Article No.: FBHEAL-1029

Pictogram Annex

^{*1}
Instructions for
use

^{*2}
XXXX = Identification number
of notified body responsible
for the EU type examination
and supervising ongoing
conformity

^{*3}
For single-use only,
must not be reused

^{*4}
Lot Number

^{*5}
Date of
Manufacture

^{*6}
Manufacturer

^{*7}
Expiration
Date

^{*8}
Sterilized using
irradiation

^{*9}
Avoid humidity

^{*10}
Keep away from
direct sunlight

^{*11}
Disposal
according to
legislation

^{*12}
Medical Device

^{*11}
Contains
LATEX proteins

^{*14}
Do not use if
package is
damaged

English

Open the pouch containing the ultrasound cover. The pouch is sterile on the inside and non-sterile on the outside. Once the pouch is torn open, simply drop its content on a sterile surface. Now wear your gloves and use a non-sterile gel to apply a little of it on the lens of your transducer. Then, pick up the cover over your dominant hand and turn it inside out so your hand is now on the inside of the cover. Grab the transducer, and gently unfurl the rest of the ultrasound cover. Secure the cover with an elastic band around the "neck" of the transducer.

*1) Instructions for use, *2) XXXX = Identification number of notified Body responsible for the EU type examination and supervising ongoing conformity, *3) For single-use only, must not be reused, *4) Lot Number, *5) Date of Manufacture, *6) Manufacturer, *7) Expiration Date, *8) Sterilized using irradiation, *9) Avoid humidity, *10) Keep away from direct sunlight, *11) Disposal according to legislation, *12) Medical Device, *13) Contains LATEX proteins, *14) Do not use if package is damaged

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Disclaimer:

The product information contained in this document, including the product images and additional product material, was collected from various sources. All product information and specifications have not been independently verified by F. Bosch International Ltd. F. Bosch International Ltd is not responsible for errors or omissions in the product information. The properties of a product may change or be inaccurate on the posting or printing of the product information in the document, either in printed or online version. Buyers and users should be cautious when using or purchasing any of the products mentioned in the printed or online versions of the documents by closely examining the product packaging and the labeling prior to use. Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in placing orders and F. Bosch International Ltd disclaims all responsibility for its completeness and accuracy. © 2023 F. Bosch International Ltd.