---

MEDIZINTECHNIK  
GERMANY

**Instructions For Use**

## ***Therapin soft Active***<sup>®</sup>

**Product:** Orthopedic bandage

**Article No.:** FBHEAL-1056, 1057, 1058, 1059, 1060, 1061

## Pictogram Annex

\*1

Instructions for use

\*2

Indicates that the product meets EU safety, health and environmental protection requirements

\*3

For single-use only, must not be reused

\*4

Lot Number

\*5

Date of Manufacture

\*6

Manufacturer

\*7

Expiration Date

\*8

Non-sterile

\*9

Avoid humidity

\*10

Keep away from direct sunlight

\*11

Disposal according to legislation

\*12

Medical Device

\*13

Handle with care

## English

Keep the rolled part of the bandage above the injury and the unrolled part below the injury. Begin by wrapping twice around the injury to hold the end in place. Work up the limb, winding the bandage in spiraling turns, making sure that each new layer covers half of the previous one. Finish by wrapping the bandage around once more and securing the end using a safety pin, tape or a bandage clip. Avoid covering fingers or toes when bandaging a limb so you can easily check the circulation.

\*1) Instructions for use, 2\*) Indicates that the product meets EU safety, health and environmental protection requirements \*3) For single-use only, must not be reused, \*4) Lot Number, \*5) Date of Manufacture, \*6) Manufacturer, \*7) Expiration Date, \*8) Non-sterile, \*9) Avoid humidity, \*10) Keep away from direct sunlight, \*11) Disposal according to legislation, \*12) Medical Device, \*13) Handle with care

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### Disclaimer:

The product information contained in this document, including the product images and additional product material, was collected from various sources. All product information and specifications have not been independently verified by F. Bosch International Ltd. F. Bosch International Ltd is not responsible for errors or omissions in the product information. The properties of a product may change or be inaccurate on the posting or printing of the product information in the document, either in printed or online version. Buyers and users should be cautious when using or purchasing any of the products mentioned in the printed or online versions of the documents by closely examining the product packaging and the labeling prior to use. Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in placing orders and F. Bosch International Ltd disclaims all responsibility for its completeness and accuracy. © 2023 F. Bosch International Ltd.